**CFS FLEXIBLE** provides flexible and unbreakable prosthetic denture partials, comprised of a nylon base thermoplastic, bio-compatible, inhalator and hypoallergenic physical properties, which are excellent materials for removable restorations.

**CFS FLEXIBLE** is your best option in prosthetic partial dentures. It combines flexibility, comfort, durability, and quality all into one.

**COMFORT & FLEXIBLE SYSTEMS** Certifies and guarantees our unbreakable product for the life of your prosthetic partial, under adequate use and strict elaboration process by your dental laboratory. This certificate is proof of authenticity and guarantees the quality of this material provided. All final products, resulting from this material are subject to the quality craftsmanship of your dental laboratory. Please be advised, that a highly knowledgeable and capable processing dental laboratory must be used for optimum results.

In order to maintain your new restoration in optimal condition, **CFS FLEXIBLE** recommends following the guidelines below:

1. Continue brushing your teeth and gums, indicated by your dentist.
2. Hygienically cleanse your **CFS FLEXIBLE** by using **CFS Denture Cleaner** in the following manner:

- Stir 3 tablespoons of **CFS Denture Cleaner** powder into 1 cup of warm water. Allow solution to rest for 3 minutes. Soak dental piece for 15 minutes daily to clean.

- Remove loosened particles with a soft brush. Rinse thoroughly under running water. Extremely dirty dentures may require an overnight soaking. Solution should last 7-10 days.

- Do not use bleach, tooth paste, or any other abrasive solutions that may have chemical effects to the texture, color & shine of your prosthetic restoration.

3. It is recommended that you keep your **CFS FLEXIBLE** restoration in water when not in use.

**IMPORTANT:** Dentures containing soft liners should not be soaked for more than 15 minutes per cleaning.

Comfort & Flexible Systems distributed by:

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